
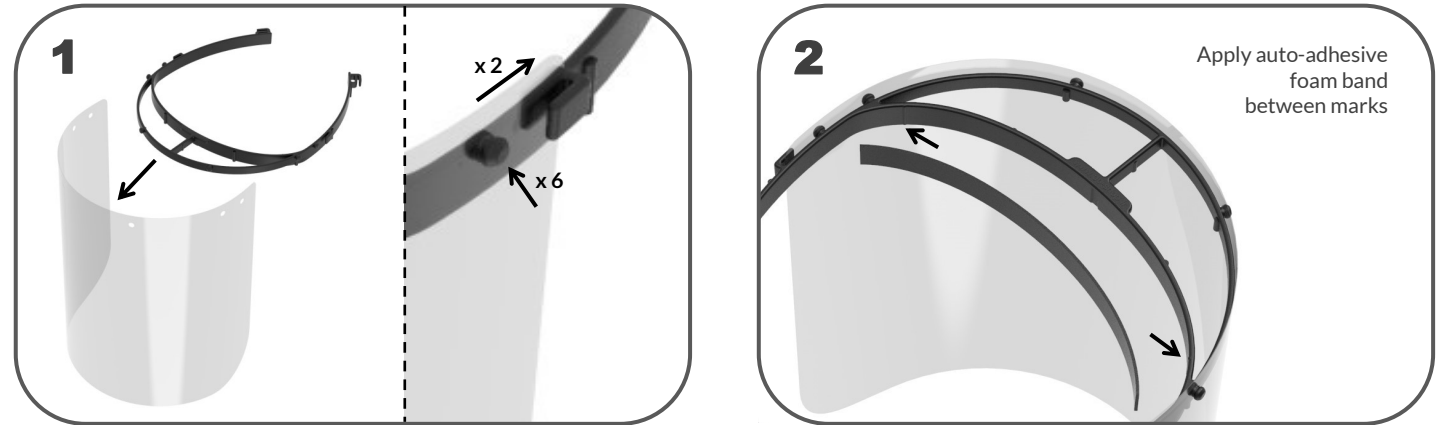


PROTECTIVE FACE SHIELD

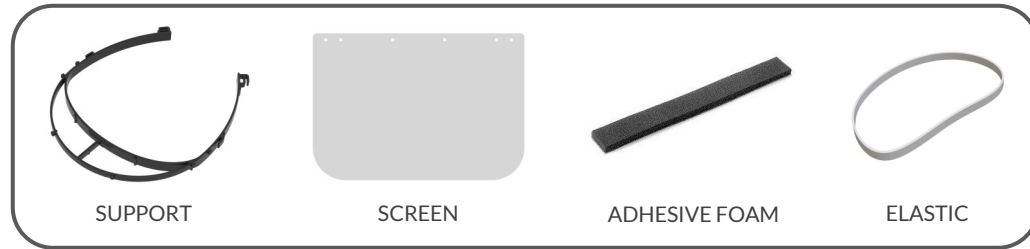


PROTECTIVE VISOR IN A KIT FORM

This non-sanitary device does not meet any certification, it provides basic protection against projections and is recommended with the wearing of a mask.

This product is not a personal protective equipment P.P.E.

It remains under the responsibility of its user.

Elastic - 2 fastening methods

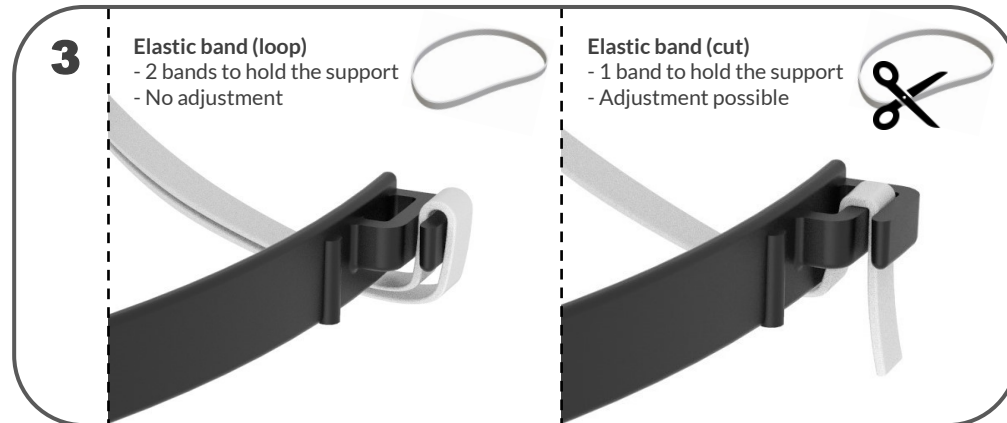

Each visor is for an individual use. Let the user assemble its own device in order to limit the contacts. To guarantee its efficiency it should not be shared.

The screen, easy to replace, should be cleaned regularly with warm soap water at 60° or with antiseptic cloth or any other soft fabric. Let it completely dry before use. The use of a soft fabric can make tiny scratches that may impair visibility. In that case it is mandatory to change the screen.

This device can generate distortions and must not be used while driving a vehicle.

DO NOT THROW ON THE PUBLIC HIGHWAY

www.db-shop.fr
info@db-shop.fr